


Model 16 - Buggy/Wheelchair Harness

Designed to offer postural support and a certain degree of restraint to children and teenagers when seated in a buggy (such as Maclaren Major Buggy or Convoid Cruiser) or folding wheelchairs.

The chest pad and shoulder straps support the trunk. The crotch strap which should always be worn, helps prevent the passenger from sliding down in the chair.


The harness will fold up with the chair when not in use.

It is not a safety harness

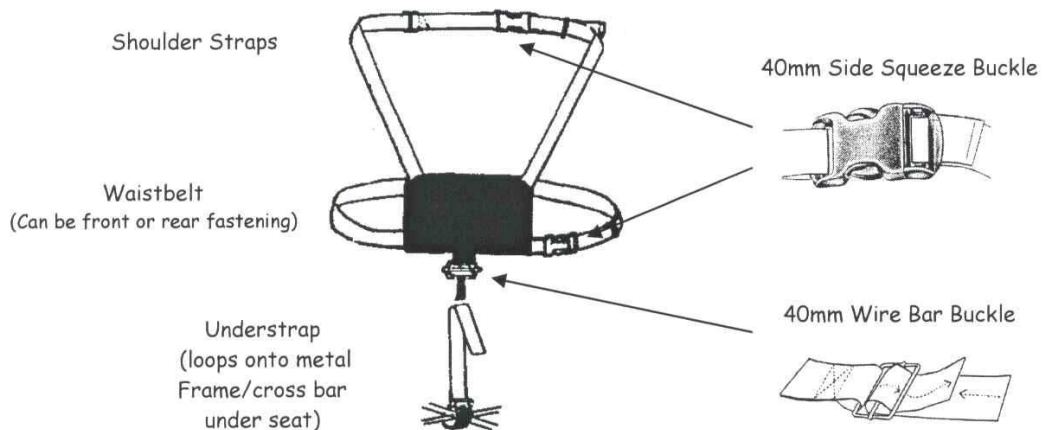


Harness fitted on a buggy.

(Can also be fitted as illustrated below)

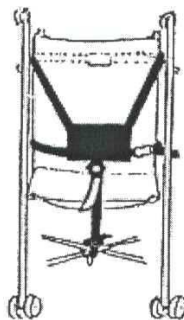


Top strap loops
Over chair back


Waistbelt passes round
child & chair back

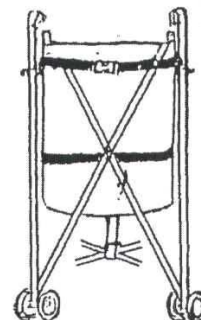

Harness fitted on a Wheelchair.

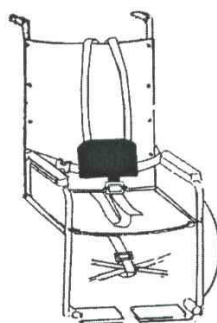

Shoulder straps loop
round waistbelt

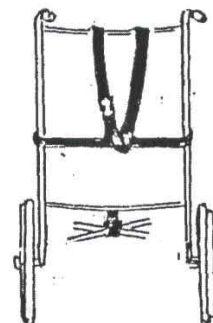

Crelling Harnesses For Disabled Ltd.

12 Crescent East, Thornton-Cleveleys, Lancashire, FY5 3LJ

Tel: 01253 852298 Fax: 01253 821780 Email: info@crelling.com Website: www.crelling.com