

Model 17

'SLIP-ON' BELT AND BRACES


A simple belt which is looped over the wheelchair backrest and simply pulled tight against the framework.

The 'slip-On' was designed with economy and simplicity in mind.

It can be fitted or removed quickly and is easily transferable from one chair to another.

This belt can be transformed into a fuller supportive harness by adding the 'slip-on' shoulder straps which fix onto the back sleeve of the belt - at rear of seat.

NOT suitable for use on folding chair.

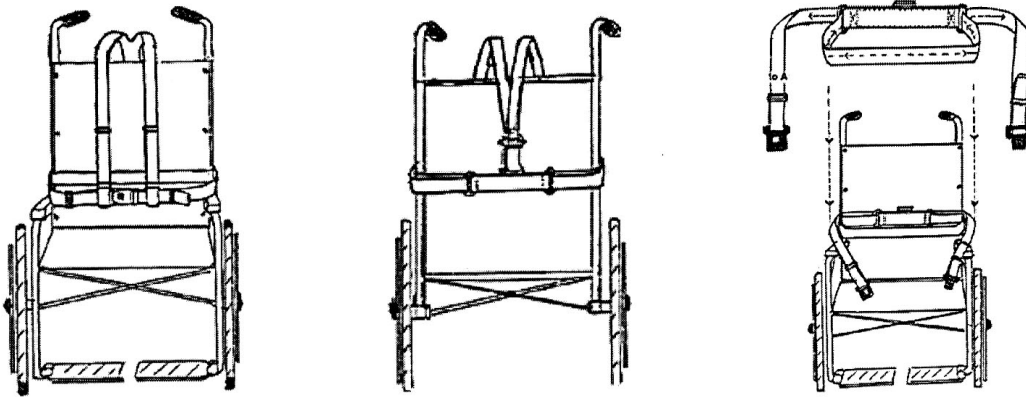

Choice of buckles available.



Sea Sure Lever Buckle



Side Squeeze Buckle

Fitting Instructions

1. Loop the waistbelt over the back of the wheelchair like a 'Slip-knot' & fasten onto the metal framework - pulling it to grip tightly around the chair.
2. Sit patient in the chair & bring the waistbelt around to fasten securely at the front waistbelt buckle.
3. Adjust the belt to fit securely yet comfortably using the slide-adjustment to one side and by pulling the loose-end of webbing to tighten the other.
4. To fit the shoulder straps - take the tail-end of the webbing at the bottom of the shoulder straps & thread it through the metal slide attached to the centre back of the PVC sleeve at the chair back. Bring the straps up the chair back, over the patient's shoulder & down over the chest to loop over either side of the waistbelt buckle.
5. Tighten the looped-ends onto the belt by adjusting the slides.
6. Now adjust all the straps so that they are tight yet comfortable for the patient.

Crelling Harnesses For Disabled Ltd.

12 Crescent East, Thornton-Cleveleys, Lancashire, FY5 3LJ

Tel: 01253 852298 Fax: 01253 821780 Email: info@crelling.com Website: www.crelling.com