

MODEL 28

Designed to offer postural support and a certain degree of restraint to children and adults to keep them in an upright sitting position.

Fits most types of vehicle seating including **single, double seats & some child booster seats.**

To fit the harness access is required between the seat back and seat cushion.

It is not a safety harness & should be worn in conjunction with the vehicle safety belt.



This harness is available with a choice of buckle on the waist belt:-



Choose 'Press Release' buckle for passengers who need postural support.

Cat. Ref: 28



Choose 'Steel Safety' buckle for those passengers with challenging behaviour or inquisitive fingers.

Cat. Ref: 28SB

NB. This harness includes a Padded Cross Strap free of charge.

**ONE SIZE ONLY - 4 Years to Adult
(Approx. 15 Kg Upwards)**

This harness has recently been modified so that one size harness fits all.


These modifications will also allow for small children travelling in booster seats with height adjustable headrests such as the Britax Adventure.

Britax Adventure



Optional Extra's

A **Cross Strap** can be fitted to hold the shoulder straps together & help prevent them from slipping off the shoulders. Also useful for those passengers who may try to separate the shoulder straps in order to escape.

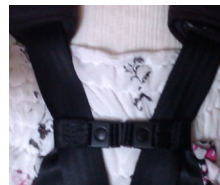

Cross Strap



Padded Cross Strap

Model 28 can be used on both single & bench seats.

Crelling Harnesses For Disabled Ltd.

12 Crescent East, Thornton-Cleveleys, Lancashire, FY5 3LJ

Tel: 01253 852298 Fax: 01253 821780 Email: info@crelling.com Website: www.crelling.com

MODEL 28 FITTING INSTRUCTIONS

1. Place the PVC backing of the harness against the front of the seat back.
2. Pass the two top vertical fixing straps up and over the seat back.
3. Pass the tail-ends of the vertical fixing down through the gap between the seat back and seat cushion and bring them up



50mm Aluminium Bar Buckle



50mm Sliplok slide

4. Position the body harness so that the waist belt will be in line with the waist.
5. Tighten the vertical straps to fit the seat back as securely as possible and pass the loose ends through the Sliplok slide.
6. With the harness unfastened at the waist belt sit the passenger on the understrap which is lying on the seat cushion and bring their arms through the shoulder straps.
7. Now bring the understrap up between the legs and loop it back onto the waistbelt over the buckle.
8. Connect the waistbelt buckle to fasten the harness.
9. Adjust all straps to fit tightly yet comfortably by pulling the tail-ends of strapping through the adjuster buckles.
10. To remove the passenger from the harness - Unfasten the buckle on the waistbelt at the centre-front.



40mm Adjuster Buckle

Squeeze Adjuster to release webbing



Standard Buckle



Steel Safety Buckle

Model 28 harness has passed dynamic Impact testing to ISO Draft Standard 10542 Pulse and complies with the VSE 87/1 voluntary code of practice.

IMPORTANT:

BELTS AND HARNESSES SHOULD BE REPLACED IF THEY SHOW ANY SIGN OF FRAYING OR CUTS, AND ESPECIALLY IF THEY HAVE BEEN INVOLVED IN AN ACCIDENT.