


Model 3 Waistbelt

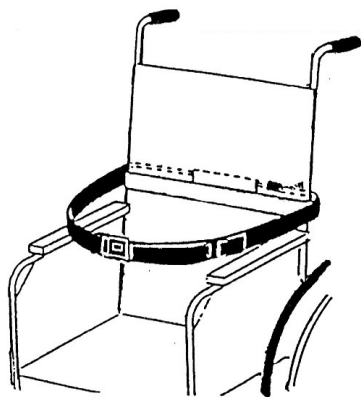

Consists of a waistbelt with an inner chair attachment strap.
Holds the occupant in wheelchair, easy chair, scooter or single vehicle seat.


Available as a rear fastening belt for the more confused patient

Available in two sizes for adults or children

Choice of buckles available.



Sea Sure Lever Buckle

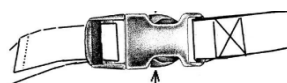

Side Squeeze Buckle



Press Button Vehicle Buckle



Steel Safety Buckle

03A Adult Waistbelt - Sea Sure Lever

03B Adult Waistbelt - Side Squeeze

03C Adult Waistbelt - Vehicle

03SB Adult Waistbelt - Steel Buckle

03E Jnr-8yrs W/Belt - Side Squeeze

03F Jnr-8yrs W/Belt - Vehicle

03J/SB Jnr-8yrs W/Belt - Steel Buckle

Fitting Instructions

1. The inner chair strap is loosened enough to pass over the chair back & positioned at the patient's waist level. Tighten the strap to fit securely.
2. Bring the waistbelt around the patient's waist & adjust to fit tightly yet comfortably.
3. To release the patient simply press release the waist belt buckle at centre-front.

The belt can be left secured to the chair when not in use.

Crelling Harnesses For Disabled Ltd.

12 Crescent East, Thornton-Cleveleys, Lancashire, FY5 3LJ

Tel: 01253 852298 Fax: 01253 821780 Email: info@crelling.com Website: www.crelling.com