

# MODEL 32

Vehicle harness designed to restrain & support children and adult passengers with learning difficulties or challenging behaviour.

Fitted with the Steel Safety buckle to the chest & waist straps as standard.

Only suitable for **single seats** or **split backed bench seats**.

**It is not a safety harness & should be worn in conjunction with the vehicle safety belt.**



**Model 32 is available in two sizes:**

3 yrs to 11 yrs      32A

12 yrs to Adult      32B

This harness is fitted with the Steel Safety Buckle to the chest & waist straps as standard.



Rear view of Model 32

The three illustrations below are showing the body harness of Model 31 which has vertical fixing straps. NB. Model 32 has the same body harness but has horizontal fixing straps.



1. Pass the two horizontal fixing straps around the back rest & fasten securely behind the seat back.

Pass the loose ends through the plastic Sliplock slides.

2. Lift the shoulder straps & PVC chest pad over the passenger's head & position it on their chest.

Adjust the shoulder straps to fit tightly & securely.

3. Bring the two horizontal straps around the child's chest & waist. These must be threaded through the vertical webbing loops on either side of the chest pad.

Loop the understrap over the waistbelt buckle.

Adjust all straps to fit tightly & securely.

## Crelling Harnesses For Disabled Ltd.


12 Crescent East, Thornton-Cleveleys, Lancashire, FY5 3LJ

Tel: 01253 852298 Fax: 01253 821780 Email: [info@crelling.com](mailto:info@crelling.com) Website: [www.crelling.com](http://www.crelling.com)


# STEEL SAFETY BUCKLE

This buckle has proven extremely successful when fitted to our harnesses as an alternative to the standard 'quick release' buckle.

It helps prevent those passengers with inquisitive fingers, challenging behaviour or learning difficulties from releasing the harness buckle.

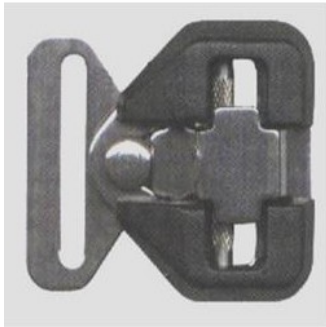

40mm Buckle

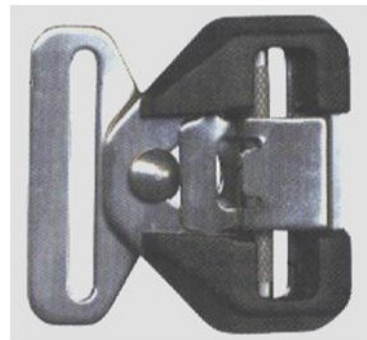

50mm Buckle



The buckle operates in reverse to the standard vehicle buckles as it is opened by raising a spring loaded lever or over-catch and **will not press release**.

The spring loaded over-catch locks the buckle when assembled and prevents accidental opening.

There is a knurled sliding bar for adjustment designed to prevent webbing slip.

The buckle is made from Type 316 Stainless Steel and Nylon 6 and is assembled with tamper proof screws.

40/45mm buckle - B/Strain 630 kgs.

50mm buckle - B/Strain 500 kgs

We recommend that the buckle instructions provided with each harness are displayed in a prominent position in the vehicle - to instruct the user how to release the buckle mechanism in the event of an accident.



## Emergency Seat Belt Cutter

We strongly recommend that a seat belt cutter is kept in the vehicle as a safety precaution.

It is perfectly safe as the blade is encased.

**Cat. Ref: SK**