


Model 7

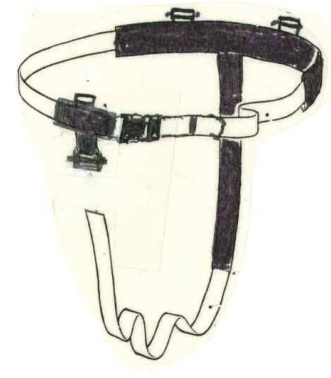

To hold confused patients or those with challenging behaviour when seated in most wheelchairs, easy chairs and semi-statics.

Rear fastening harness with PVC lined groin strap in case of incontinence etc.

The harness consists of detachable shoulder straps, waist belt and waterproof groin strap.

One size only.

Adult Size

Fitted with Side Squeeze buckle to waist belt.

Cat. Ref: 07B

Optional Extra

A **Thigh strap** can be ordered as an optional extra to an understrap - Cat. Ref: THIGH

This is a single 50mm strap fitted with a Side Squeeze buckle which passes underneath the seating & fastens around the thigh's.

Fitting Instructions

1. Unfasten the shoulder straps from the front PVC sleeve.
2. Place the T-shaped harness in front of the patient & bring the waistbelt around the patient & the wheelchair & fasten around the back of the wheelchair with the side-squeeze buckle.
3. Pass the groin strap between the patient's legs & underneath the seat to fasten it to the sliding bar buckle on the back sleeve of the waist belt.
4. Finally bring the shoulder straps up & over the wheelchair back & down over the patients shoulders & chest to fasten to the two side-squeeze buckles at the front waistbelt PVC sleeve.
5. Adjust all straps to fit the patient tightly yet comfortably.

Crelling Harnesses For Disabled Ltd.

12 Crescent East, Thornton-Cleveleys, Lancashire, FY5 3LJ

Tel: 01253 852298 Fax: 01253 821780 Email: info@crelling.com Website: www.crelling.com