
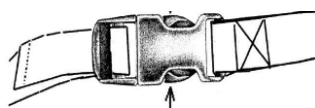


Model 8

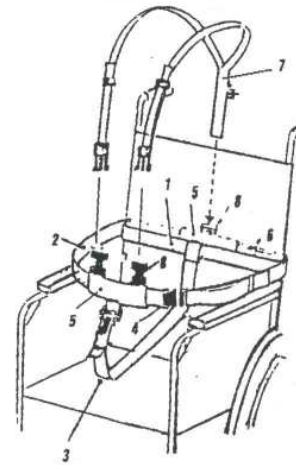

Side Squeeze Buckle

1. Chair Strap
 2. Waistbelt
 3. Detachable Crotch Strap
 4. PVC sleeves
 5. Waistbelt and understrap buckles
 6. Shoulder Straps
 7. Shoulder Strap Buckles
- Comfort pads are available by request.

A full supportive children's harness consisting of waistbelt, shoulder straps and **detachable understrap**.

The child **sits directly on the understrap** to hold them in an upright sitting position.
(If you would prefer the understrap to be passed underneath the seat please see Model 2).

One size only. Fits most children's wheelchairs, buggies & dining chairs etc.

Fitting Instructions

The harness will be supplied ready assembled.

1. Unfasten the groin strap & the shoulder straps from the front PVC sleeve.
2. Loosen the inner chair strap (coloured webbing) enough to pass over the chair back & position at the patient's waist level. Tighten to fit securely around the chair back.
3. Unfasten the waistbelt buckle at the front & position the groin strap so that it is lying on the seat. The harness is now ready for the patient to get into the wheelchair.
4. With the patient seated in the wheelchair, bring the waistbelt around their waist & fasten the side-squeeze buckle. Adjust waistbelt to fit tightly yet comfortably.
5. Fasten the groin strap to the buckle at the bottom of the PVC sleeve & adjust accordingly.
6. Finally bring the shoulder straps up & over the wheelchair back & down over the patients shoulders & chest to fasten to the two side-squeeze buckles at the front waistbelt PVC sleeve.
7. Adjust all straps to fit the patient tightly yet comfortably.

Crelling Harnesses For Disabled Ltd.

12 Crescent East, Thornton-Cleveleys, Lancashire, FY5 3LJ

Tel: 01253 852298 Fax: 01253 821780 Email: info@crelling.com Website: www.crelling.com