



STEEL SAFETY BUCKLE

This buckle has proven extremely successful when fitted to our harnesses as an alternative to the standard 'quick release' buckle.

It helps prevent those passengers with inquisitive fingers, challenging behaviour or learning difficulties from releasing the harness buckle.

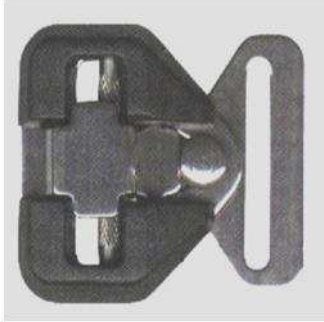

40mm Buckle

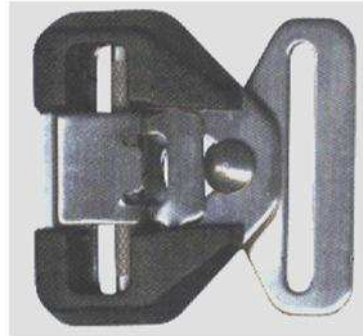

50mm Buckle



The buckle operates in reverse to the standard vehicle buckles as it is opened by raising a spring loaded lever or over-catch and **will not press release**.

The spring loaded over-catch locks the buckle when assembled and prevents accidental opening.

There is a knurled sliding bar for adjustment designed to prevent webbing slip.

The buckle is made from Type 316 Stainless Steel and Nylon 6 and is assembled with tamper proof screws.

40/45mm buckle - B/Strain 630 kgs.

50mm buckle - B/Strain 500 kgs

We recommend that the buckle instructions provided with each harness are displayed in a prominent position in the vehicle - to instruct the user how to release the buckle mechanism in the event of an accident.



Emergency Seat Belt Cutter

We strongly recommend that a seat belt cutter is kept in the vehicle as a safety precaution.

It is perfectly safe as the blade is encased.

Cat. Ref: SK

TRANSPORT OF SPECIAL NEEDS PASSENGERS

Disabled children and adult passengers travelling in vehicles should always wear a safety belt or special restraint during transport. Our range of vehicle harnesses are not designed to replace the safety harness - but work in conjunction with it to offer both safety plus support and in some cases a certain degree of restraint.

- The existing inertia *seat* belt fitted in the vehicle may not offer sufficient support to the more severely disabled and it may be necessary to provide an extra restraint to hold the passenger in an upright sitting position, in combination with the seat belt.
- Small children should be raised by a booster cushion' so that the restraint fits better and the child more comfortable during transport.
- Front seat disabled passengers in vehicles, frequently fall sideways out of the diagonal chest strap especially going around corners or sudden braking. Two vertical static shoulder straps would offer additional support.
- The combination of the existing car safety belt and the added special restraint offers added security and ensures that the disabled passenger will be well protected during transport.
- If no safety belt is available, the disabled person should be given some support or protection in a Disabled Person's Restraint.



Illustrating one of our harnesses fitted in conjunction with the existing vehicle safety belt.

BOOSTER CUSHIONS

Booster cushions can be used in conjunction with all our range of vehicle harnesses.

This helps to raise the child to the correct height for wearing the vehicle safety belt.

For current laws and guidelines please visit:

www.rospa.com

or

www.childcarseats.org.uk

Please note that all our range of vehicle harnesses are designed for special needs passengers and should be worn in conjunction with the existing safety belt in the vehicle.

Crelling Harnesses For Disabled Ltd.

12 Crescent East, Thornton-Cleveleys, Lancashire, FY5 3LJ

Tel: 01253 852298 Fax: 01253 821780 Email: info@crelling.com Website: www.crelling.com