

For adults or standard size wheelchairs – Choose 'Model 1' – The waist belt is fitted with the 50mm Side Squeeze buckle.
For children or Junior size wheelchairs – Choose 'Model 2' – The waist belt is fitted with the 40mm Side Squeeze buckle.

Available in two sizes:

☐ **Model 1** - 9 years - Adult

☐ **Model 2** - Up to 8 years.

A fully supportive wheelchair harness consisting of a waist belt, shoulder straps, and an under strap. It is designed to help maintain the occupant in an upright seated position. The under strap passes underneath the base of the wheelchair seat.

Note: If there is access between the wheelchair backrest and the seat base, it may be possible to thread the under strap through this gap, allowing the occupant to be seated directly on it if preferred.

Important: Access between the wheelchair backrest and seat base is required to fit this harness.

Velcro Comfort Sleeves. (Included)

Velcro comfort sleeves are supplied with each harness. These can be fitted around the shoulder straps and Under strap where required.

Emergency Seat Belt Cutter (Optional Extra)

We strongly recommend that a seat belt cutter is kept in the vehicle as a safety precaution for those harnesses fitted with a Steel Safety Buckle.

A simple but effective device for ensuring the occupants of a car are able to exit quickly and safely, particularly in an overturned car.

IMPORTANT: This harness is not classed as a safety harness.

The vehicle safety belt MUST be worn over the top of our harness unless exempt for medical reasons.

The combination of the existing safety belt & the secondary harness offers added security for the passenger. For non UK use: Please ensure you check and are compliant with your own countries regulations / laws for child seat and harness use.

Crelling Harnesses For Disabled Ltd.

12 Crescent East, Thornton-Cleveleys, Lancashire, FY5 3LJ

Tel: 01253 852298, Email: info@crelling.com Website: www.crelling.com

Model 1 & 2 Fitting Instructions.

The harness is supplied ready-assembled.

Preparation

1. Unfasten the under strap and both shoulder strap buckles. Loosen the inner chair strap (coloured differently from the black webbing).
2. Pass the inner chair strap around the back rest and position it at the patient's waist level. Fasten the buckle and tighten the strap by pulling the free webbing end from the buckle and then taking up the slack with the slide.
3. Unfasten the front waist belt buckle. The harness is now ready for the occupant.

Fitting

1. Seat the occupant in the wheelchair.
2. Bring the waist belt around the occupant's waist and fasten it tightly yet comfortably.
3. Route the understrap down the back rest, under the seat base, up between the patient's knees, and connect to the side-squeeze buckle at the bottom of the waist belt PVC sleeve. Adjust tightly yet comfortably. Optional: If a gap exists between the backrest and seat, the under strap may be threaded through it so the patient sits directly on the strap.
4. Bring the shoulder straps up over the wheelchair back, down over the patient's shoulders and chest, and fasten them to the two side-squeeze buckles on the waist belt. Adjust tightly yet comfortably.

Finally, check that all straps are secure and that the occupant is comfortable.

IMPORTANT: This harness is not classed as a safety harness.

The vehicle safety belt MUST be worn over the top of our harness unless exempt for medical reasons.

The combination of the existing safety belt & the secondary harness offers added security for the passenger. For non UK use: Please ensure you check and are compliant with your own countries regulations / laws for child seat and harness use.