

Model 13

Designed to offer postural support and a certain degree of restraint to children and adults to keep them in an upright sitting position.

Fully rear fastening harness designed for those passengers with challenging behaviour or learning difficulties. This harness can be used on some wheelchairs and easy chairs but when used in vehicles will only fit single seats with easy access around the back rest of the seat & underneath the base of the seat cushion.
i.e. people carriers, mini-buses or the front passenger seat.



☐ **13H** - 7 yrs to Adult
☐ **13HSTEEL3** - 7 yrs to Adult



☐ **13J** - 2 yrs to 6 yrs
☐ **13JSTEEL3** - 2 yrs to 6 yrs



☐ **13W** - 7 to Adult
☐ **13WJ** - 2 to 6 yrs

Key Features

- Press release buckle as standard
- Fits most vehicle seats
- Suitable for children and adults

Not suitable for the rear seat of most standard hatchback vehicles.

Velcro Comfort Sleeves. (Included)

Velcro comfort sleeves are supplied with each harness. These can be fitted around the shoulder straps and Under strap where required.



Emergency Seat Belt Cutter (Optional Extra)

We strongly recommend that a seat belt cutter is kept in the vehicle as a safety precaution for those harnesses fitted with a Steel Safety Buckle. A simple but effective device for ensuring the occupants of a car are able to exit quickly and safely, particularly in an overturned car. **Model: SKBRANDED**



IMPORTANT: This harness is not classed as a safety harness.

The vehicle safety belt MUST be worn over the top of our harness unless exempt for medical reasons.

The combination of the existing safety belt & the secondary harness offers added security for the passenger. For non UK use: Please ensure you check and are compliant with your own countries regulations / laws for child seat and harness use.

Crelling Harnesses For Disabled Ltd.

12 Crescent East, Thornton-Cleveleys, Lancashire, FY5 3LJ
Tel: 01253 852298, Email: info@crelling.com Website: www.crelling.com

Model 13 Fitting Instructions

We recommend two people fit this harness to ensure the occupant stays in the seat whilst the harness is secured and adjusted.

1. Place the PVC chest pad against the occupant's chest.
2. Pass the upper and lower fixing straps around the occupant and fasten them behind the seat back.
(lower strap at waist level and the top strap at chest level to hold the occupant securely in an upright sitting position.)
3. Pass the shoulder straps over the occupant's head and shoulders
4. Position the buckle tongue behind the seat back.
5. Adjust the shoulder straps so that the buckle remains out of the occupants sight and reach.
6. Pass the understrap between the occupant's knees, down underneath the seat base and up behind the seat back to connect to the shoulder strap buckle, connect the buckle.
7. Adjust all straps to fit securely yet comfortably.



Steel Safety Buckle



This buckle has proven extremely successful when fitted to our harnesses as a secure alternative to the standard 'quick release' or 'press release' buckle.

It is recommended for passengers who can unfasten conventional press-release buckles, or in situations where other passengers on the vehicle might release them. Children and adults with learning difficulties or autism find this buckle more complicated to open.

Its design helps prevent those with inquisitive fingers, challenging behaviour or learning difficulties from releasing the harness.



The Steel buckle operates in reverse to the standard vehicle buckles as it is opened by lifting a spring loaded lever or over-catch and **will not press release!**

The spring loaded lever/over-catch locks the buckle when assembled and prevents accidental opening. There is a knurled sliding bar for adjustment designed to prevent webbing slip.

The buckle is made from Type 316 Stainless Steel and Nylon 6 and is assembled with tamper proof screws.

For Steel buckle variants please look at Model 13J/STEEL3 (up to 6 yrs) or Model13H/STEEL3 (7yrs-adult)

Sliplock Slide



Pass each loose webbing end back through the sliplock slide to help prevent the straps from loosening during use.

Booster cushions



The Model 13 can be used with a booster cushions but not high-backed booster seats..

IMPORTANT: Our harnesses must NOT be used with child car seats already fitted with the 5 point harness.

IMPORTANT: This harness is not classed as a safety harness.

The vehicle safety belt MUST be worn over the top of our harness unless exempt for medical reasons.

The combination of the existing safety belt & the secondary harness offers added security for the passenger. For non

UK use: Please ensure you check and are compliant with your own countries regulations / laws for child seat and harness use.