

For children – Model 3BJ / Model 3SBJ

For children – Model 3B / Model 3SB

Available in two sizes with two buckle choices:

- ☐ **Model 3BJ** - Up to 8 years. With Side-squeeze buckle
- ☐ **Model 3SBJ** - Up to 8 years. With Steel buckle

- ☐ **Model 3B** - 9 years - Adult. With Side-squeeze buckle
- ☐ **Model 3SB** - 9 years - Adult With Steel buckle

A fully supportive wheelchair harness consisting of a waist belt and inner chair strap.
It is designed to help maintain the occupant in an upright seated position.

Important: Access between the wheelchair backrest and seat base is required to fit this harness.

Velcro Comfort Sleeves. (Included)

Velcro comfort sleeves are supplied with each harness. These can be fitted around the shoulder straps and Under strap where required.

Emergency Seat Belt Cutter (Optional Extra)

We strongly recommend that a seat belt cutter is kept in the vehicle as a safety precaution for those harnesses fitted with a Steel Safety Buckle. A simple but effective device for ensuring the occupants of a car are able to exit quickly and safely, particularly in an overturned car.

IMPORTANT: This harness is not classed as a safety harness.

The vehicle safety belt MUST be worn over the top of our harness unless exempt for medical reasons.

The combination of the existing safety belt & the secondary harness offers added security for the passenger. For non UK use: Please ensure you check and are compliant with your own countries regulations / laws for child seat and harness use.

Crelling Harnesses For Disabled Ltd.

12 Crescent East, Thornton-Cleveleys, Lancashire, FY5 3LJ

Tel: 01253 852298, Email: info@crelling.com Website: www.crelling.com

Model 3 Fitting Instructions.

The harness is supplied ready-assembled.

1. Loosen the inner chair strap sufficiently to pass it around the wheelchair backrest. Position the strap at the occupant's waist level, fasten the buckle, and tighten the strap securely.
2. Bring the waist belt around the occupant's waist and adjust it until it is secure yet comfortable.
3. To release the occupant, press the centre-front waist belt buckle.
4. Finally, check that all straps are secure and that the occupant is comfortable.

IMPORTANT: This harness is not classed as a safety harness.

The vehicle safety belt MUST be worn over the top of our harness unless exempt for medical reasons.

The combination of the existing safety belt & the secondary harness offers added security for the passenger. For non UK use: Please ensure you check and are compliant with your own countries regulations / laws for child seat and harness use.