

STAIRLIFT

Designed to offer postural-support and maintain a good upright sitting position for those with special needs when using a stair-lift.

Available in various combinations from a simple waist-belt to a full supporting 5-Point harness.

Available in Adult and Junior sizes. Junior size is available as a 5-Point harness:



☐ STAIR 3 - 2-Point harness



☐ STAIR 2 - 4-Point harness



☐ STAIR 5 - 5-Point harness



☐ STAIR J - 5-Point Junior

Designed to offer postural-support and maintain a good upright sitting position for those with special needs when using a stair-lift.
For temporary fitting harnesses, please see [Model 19](#), [Model 27](#) or the Aqua harness.
If only a waist belt is required, please consider Model 3.

IMPORTANT:

- **This is a permanently fitted harness which is secured to the stairlift seat using anchorage brackets. For guidance on suitable anchorage points and fixing components, contact the stairlift manufacturer or supplier, as these will vary depending on the stairlift model.**

Velcro Comfort Sleeves. (Included)

Velcro comfort sleeves are supplied with each harness. These can be fitted around the shoulder straps and Under strap where required.



Crelling Harnesses For Disabled Ltd.

12 Crescent East, Thornton-Cleveleys, Lancashire, FY5 3LJ

Tel: 01253 852298, Email: info@crelling.com Website: www.crelling.com

Stairlift Fitting Instructions.

The harness is fitted using steel anchorage lugs, which must be securely fixed to suitable anchorage points on the stairlift seat. The location and method of fixing will depend on the stairlift model. Consult the stairlift manufacturer or supplier if you are unsure about suitable anchorage points.

Fitting

1. Seat the occupant comfortably on the stairlift and bring each arm through the shoulder straps.
2. Bring the waist belt around the occupant and adjust it to fit securely and comfortably.
3. If applicable, pass the understrap between the occupant's legs. Release the waist-belt buckle, place the understrap loop over the buckle, then refasten the waist belt.
4. Adjust the understrap by pulling the loose end until it fits securely and comfortably.
5. Adjust the slides on the shoulder straps until the straps fit securely and comfortably.
6. Finally, check that all straps are secure and that the occupant is comfortable.



IMPORTANT: This harness is not classed as a safety harness.

For use outside the UK, ensure that the installation and use of the harness comply with all applicable local regulations.